

Instruction for use

HYDROCYN aqua®

INGREDIENTS

Active – HOCl (0.003%)

Inactive – NaOCl + NaCl (< 0.1%)

Water – 99.9%

INDICATION

HYDROCYN aqua® is indicated as a wound wash, debridement agent & cleaning solution for acute and chronic wounds in stage I-IV ulcers, venous stasis & diabetic ulcerations, surgical wounds, burns (1st - 2nd degree) and bed sores. It is also can be used in moistening, wound dressing during cleaning and irrigating wounds. HYDROCYN aqua® has ancillary antimicrobial properties to accelerate wound healing.

DIRECTION FOR USE

HYDROCYN aqua® is ready for use with no mixing or dilution required. Apply directly to the affected area. Use at each wound dressing change. Saturation of the dressing with HYDROCYN aqua® will assist with dressing removal.

NOTE

Soak/saturate dressing with HYDROCYN aqua® at each dressing change, as and when required. Immersion is allowed for maximum 15 minutes, daily, during initial treatment.

STORAGE

HYDROCYN aqua® should be stored in a cool dry place (recommended storage below 30°) away from direct sunlight/extreme weather condition in its original container. After use, the container should be closed again, before storing.

STABILITY

Product is stable for 24 months if unopened and in original pack. However, it only can be used within 30 days once the bottle is opened. Stability is affected if the bottle is not stored or closed properly after use.

WARNINGS AND POINTS TO NOTE

Do not use the product if the tamper proof seal is damaged or tampered with. Bottle has to be discarded if signs of contaminants are seen. Do not dilute or use the product in combination with other topical solution unless with the advice of a health care specialist. Consult your health care specialist if wound persists to deteriorate even after use of solution. Notify your health care specialist in case of any adverse effect. In case of deep or serious wounds, or if you have any other treatment concerns consult a health care professional. If signs of infection occur (increased pain, bleeding, pus) or if the wound does not show signs of healing, consult a health care professional. Do not use if hypersensitive to product components. This device is intended for “Single Patient Use”. Keep out of the reach and out of sight of children. For external use only.

RISK OF RE-USE

HYDROCYN aqua® cannot be re-used if it is poured outside the container. Always use fresh solution.

SYMBOLS USED ON LABELLING



Article Number



Batch Number



Caution



Use until year + month



Keep away from sunlight



Manufacturer



Do not use if package is damaged



Authorized representative in the European Community



OBELIS S.A
BD. GENERAL WAHIS 53
1030 BRUSSELS, BELGIUM

Manufactured by:

VIGILENZ MEDICAL DEVICES SDN BHD

308 b, Jalan Perindustrian Bukit Minyak 18,
Penang Science Park, 14100 Penang.
Tel: +604 505 4241